

# GlucorX VOYAGER

## How to import readings from USB Meters to Voyager for Windows

1. Navigate to <https://www.glucorx.co.uk/our-services/software/> and select the Voyager Download button.

**CREATE ACCOUNT**

First Name\*

Last Name\*

Male  Female

Select a date 15

Email\*

Password\*

Confirm Password\*

Thank you for signing up for GlucoRX. Please click on the following button to verify your email and get started.



Best Regards,  
GlucoRX Support Team

2. To login, Input your email address and password.

Welcome To GlucoRx  
Login

Email / User ID

Password

Forget Password

SUBMIT

Create Account

3. The user is prompted to input their Low and high blood glucose limits. This is personal to each person. If you are unsure what your limits are you should speak with your healthcare professional. (This is only required on the user's first login)

4. Plug your meter into the computer using a USB cable.

5. When "USB or PC" is displayed on the screen, select the meter name from the devices dropdown and select "Connect".

6. New readings will automatically be imported.

7. The GlucoRx Voyager portal will open in a web browser.

Thresholds	
Units of Measure	mmol
Low Limit	4.1
High Limit	9.5
High Alarm	10.5
Low Alarm	3.5



7. To log into the portal without importing any results, open GlucoRx.exe and click the "visit Portal" button.

You can also login from any web browser by going to [www.grxvoyager.com](http://www.grxvoyager.com)