

Part no.	319-4611100-XXX
Product name	機器紙卡/U-RIGHT/4611A/英文/
Spec	L148*W210mm/對1折/模造紙70P/黑
Designer	Emily
Color	 K:90  K:50  K:35

Important Notice For Operation

Dear GlucoRx U-Right HbA1c Analyser Owner:

This instruction contains important information you must know before use. Please read it thoroughly and carefully.




close cover

1. Before Measurement

- Ensure the cover is closed while initiating the system.

2. Preparing Patient Samples


- Import the lot information for each new box of cartridge.

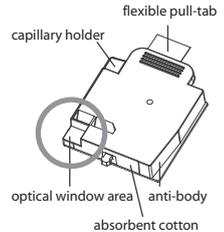

The reagent cartridge must reach room temperature before use.


- When handling the cartridge, do not touch the optical window or erroneous test results may occur.

- Too much or less amount of blood may lead to erroneous test results.

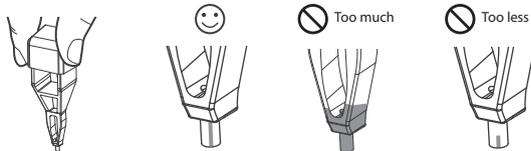


- When the capillary is filled with the sample, analysis must begin immediately (to avoid the blood drying).

- Do not pull the tab before you insert the cartridge into the cartridge compartment. If the cartridge is damaged or the flexible pull-tab is loose or missing, please discard it and replace with a new one.

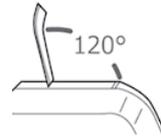


3. Analyser Operation

- Open the cover at a tilted angle of 120 degrees until a click is heard or felt. If the cover is not open at 120 degrees, the cover may close again or the cartridge may not insert completely. This may damage the Analyser.

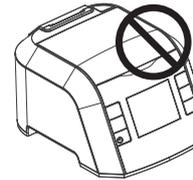

- Hold the reagent cartridge with the label facing left. The cartridge is designed to fit only one way into the system. Do not force the cartridge into the system.
- Do not open the cover during the measuring process; the rotation of the internal device may cause personal injury or damage the device.

- To place the thermal paper, the induction surface of paper roll should be facing up and be threaded into the correct direction before printing.

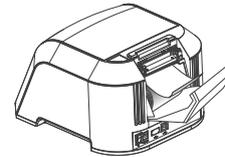

Distributed by GlucoRx Ltd.
4 Wintonlea, Monument Way West,
Woking, Surrey, GU21 5EN, UK
Website: www.glucorx.co.uk
Customer careline: 01483 755133

TaiDoc Technology Corporation
B1-7F, No. 127, Wugong 2nd Rd., Wugu Dist.,
24888 New Taipei City, Taiwan

EC REP MedNet GmbH
Borkstraße 10, 48163 Münster, Germany

